Together against COVID-19

The COVID-19 pandemic confronts us with new challenges. The well-being of our employees and retailers is our highest priority. Therefore, we strictly adhere to the WHO guidelines on social distancing, protection and prevention. However, in direct contact with customers it is not always possible to follow all guidelines. Therefore, Eagle Creek supports you with this protective face mask. If worn, you can help prevent the spread of COVID-19.

About the mask

- ▶ The protective face mask is made of two layers: the outer layer - 100% cotton, the inner layer - 95% polyester, 5% elastane.
- ▶ All face masks have SilverPlus® finish - a unique technology that prevents odor-causing bacteria. Silver releases positive ions that attach to the negative ions of bacteria, which prevents the growth of odor-causing bacteria. The silver ions protection will be effective only during wear.
- ▶ The fabric is soft and fits well on the face. Your skin will stay free from any irritation and itching no matter how long you wear this the face mask. Also, it allows for breathing easily.
- ▶ The elastic straps fit comfortably around the ears. People of all ages, shapes, and sizes can wear this reusable face mask without any discomfort.
- ▶ Most important it is reusable. The product can be washed by hand in the warm water or a washing machine at 40 °C. This face mask stays soft and durable no matter how many times you wash and dry it.

Please note

- ▶ The product offered is not a medical respirator. The product does not comply with FFP standards and is not certified.
- ▶ Wearing the mouth cover does not protect against respiratory infections. At best, it may reduce the risk of pathogen transmission to others. No guarantee is given for this. It is scientifically recommended to wear it tightly and to change it regularly when it becomes soaked.

How to wear the mask correctly?

- ▶ Hold the mouth-nose mask in both hands so that the Eagle Creek logo is visible on the inside. Now put it on your face from the front and fix the two bands behind the ears. The mask should cover your mouth and nose completely.

Safety Advice

By taking just a few simple measures, you can help protect yourself and others against infectious diseases.

- ▶ Wear a face mask.
- ▶ Wash your hands regularly and properly (at least 20 seconds) with soap and water.
- ▶ If you need to cough or sneeze, use a tissue or the crook of your arm.
- ▶ If possible, maintain at least 1,5 m distance to other people, particularly those who have a cough, runny nose, or fever.
- ▶ Ventilate enclosed spaces several times a day for a few minutes.

If you have any questions about face mask, please contact your sales rep or our customer service:

Phone +353.21.4621.471
E-Mail info@eaglecreek.ie

Please pay attention to yourself!

Your Eagle Creek Team
#FindYourUnknown

Eagle Creek Europe Ltd
Dwyer Road
Middleton, County Cork
Ireland

Companies Registration Dublin, Ireland # 350487
VAT # IE6370487D
Managing Director Oliver Messing, Roger Spatz

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Telefon +353.21.4621.473
E-Mail info@eaglecreek.ie

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Dein Eagle Creek Team
#FindYourUnknown